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(N-D231)

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INSTRUCTION MANUAL

(N - D231)

Thank you for choosing Nabtesco's dual connector. For your safety, read this manual and make sure that you fully understand the contents so that you can use the product correctly.

1 Application

This product is a connector designed to connect between components that have male pyramids.

- DO NOT use for a person who weighs over 100 kg (220lb).
- * Failure to observe the prohibitions could result in damage to the products and user injury.

Prohibition

2 The range of adjustment

The length of this connector is adjustable in the range from 60mm to 75mm. The rotational direction can be adjusted as well. (it changes length 0.5mm by 90° rotation.)


- DO NOT modify the products such as changing the maximum length (75mm).
- * Failure to observe the prohibitions could result in damage to the products and user injury.

Prohibition

3 How to Use

- After adjusting length and angle of rotation, Apply adhesive(equivalent to LOCTITE 242)on the thread part of the M5 set screw (1 part), and tighten by a specified torque.
- * If the connector is still rotated with tighten by the regulated torque, once, loose the screw and then tight the screw by the regulated torque again.
- After adjusting the angle of alignment, Apply adhesive(equivalent to LOCTITE 242)on the thread part of the M8 hexagon head socket bolt (4 part), and tighten by a specified torque.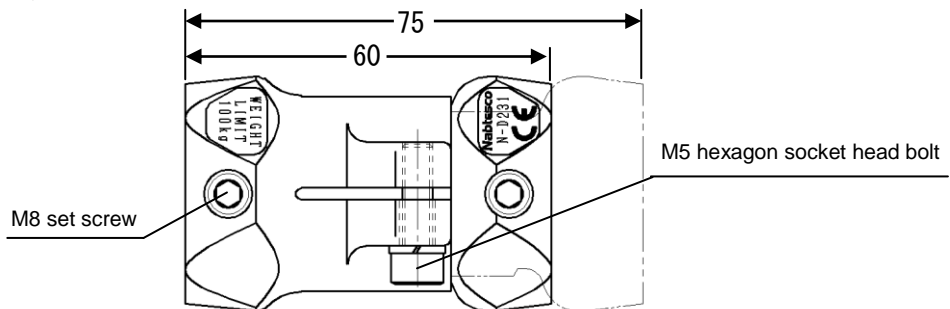

- Tighten each of the set screws or hexagon socket head bolt with the designated torque by a torque wrench.

M8 Set screw Tightening torque: $10 \pm 0.5 \text{ N} \cdot \text{m} (102 \pm 5 \text{ kg} \cdot \text{f} \cdot \text{cm})$

M5 Hexagon socket head bolt Tightening torque: $5 \pm 0.25 \text{ N} \cdot \text{m} (51 \pm 2.5 \text{ kg} \cdot \text{f} \cdot \text{cm})$

WARNING

- * If the torque used to tighten the screws is less than the designated amount, the set screws could become loose and fall out during use
- * If the torque used to tighten the screws is more than the designated amount, witch could break the connector during use.
- Apply the adhesive(equivalent to LOCTITE 242)on the thread part for anti-loosening of each screw, and tighten by a specified torque.
- * Without using adhesive, the set screws could become loose and fall out during use.

4 Warranty

- The warranty for this dual connector is valid for a period of one year from the date of purchase.
- Note that if the connector is broken or damaged due to improper use the warranty may become void even if it happens during the warranty period.

Nabtesco Corporation

Manufacturer

Nabtesco Corporation
 Accessibility Innovations Company
 Assistive Products Department
 35, Uozakihama-machi, Higashinada-ku
 Kobe 658-0024, Japan
 Phone: +81-78-413-2724 Fax: +81-78-413-2725
 URL: <http://welfare.nabtesco.com>

Authorized representative for EU countries

NABMIC B.V.
 Brouwerstraat 34, 2984 AR Ridderkerk The Netherlands
 Phone: +31-180-530-590
 Fax: +31-180-530-591