# Pipe N-P233

## Instruction Manual

### 1. Information

This product must be assembled by a certified prosthetist. Please read this document carefully before using the product and observe the safety notices. Instruct the user in the safe use of the product.

Please contact the manufacturer if you have questions about the product or in case of problems.

Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.

Please keep this document for your records.

### 2. Purpose of use and product overview

This product is a pipe intended to be assembled with the intelligent knee joint (NI-C1 series, NI-C3 series, NI-C4 series) and a prosthetic foot via a  $\varnothing$  30 mm tube-holder connector. It is for everyday life and can be used multiple times by one user.

You can adjust the length of the prosthesis and the internal rotation and external rotation of the prosthetic foot.

### 3. Product Specifications

- Maximum body weight: 100 kg (ISO 10328-P5-100kg)
- K Level (MOB): K2 Level (MOB2) ~ K4 Level (MOB4)
- Expected lifetime: 3 years
- Applicable environmental conditions
  - Storage temperature range: -20°C to +60°C (-4°F to +140°F)
  - Operating temperature range: -10°C to +40°C
  - Chemicals/liquids: Water or liquid (salt water, sweat, urine) splash
- Prohibited environmental conditions
  - Place of use: Underwater (in liquid)
  - Chemicals/liquids: Hydrochloric acid, sulfuric acid

### 4. Safety

Explanation of warning symbols

	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
	Indicates a potentially hazardous situation which, if not avoided, could result in minor or moderate injury.
	Indicates actions that you must not do when handling the product.

Strictly observe the following instructions to use the product safely.

- Tighten the pipe clamp with the specified torque of the intelligent knee joint.
- If the torque used to tighten the screws is less than the designated amount, the set screws could become loose and fall out during use.
- Adjust the alignment according to the instructions of the intelligent knee joint.

- DO NOT use the pipe with any other prosthetic knee than the intelligent knee joint.
- DO NOT use for a person who weighs over 100 kg (220 lb).
- DO NOT cut the  $\varnothing$  34 area of the pipe.

Failure to follow the instructions could cause the foot side to turn and thereby the user may fall.

- Do not leave or use the product in an environment not applicable.
- If there is any abnormality such as damage, abnormal noise, or backlash, discontinue use and take appropriate measures. (Contact the manufacturer or specialist workshop)
- Do not use the product beyond its useful lifetime.

### 5. How to use

You can use N-P233 pipe by cutting its  $\varnothing$  30 area.

- \* If you cut the  $\varnothing$  30 side to less than the insertion length of the adapter to be used, it will cause decline in pipe clamp force.


- After cutting the pipe, remove burrs on both the inner side and outer side on the end face.

**CAUTION** Failure to do so could cause you to cut your finger with burrs.

#### 5.1. How to adjust the pipe length

The length of the pipe must be adjusted by a prosthetist or a person who can handle prostheses.

- Mounting direction  
 Knee joint side:  $\varnothing$  34 pipe      Foot side:  $\varnothing$  30 pipe

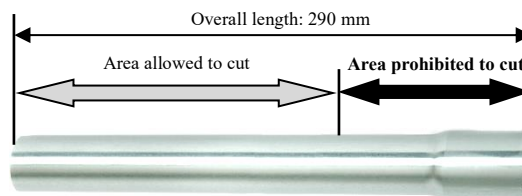#### 5.2. Cleaning

If necessary, wipe the surface with a soft cloth moistened with fresh water.

#### 5.3. Maintenance

For safety, carry out regular inspections once a year for loose screws or abnormalities.

#### 5.4. Disposal

This product is made of aluminum alloy. Observe the information provided by the responsible authorities in your country regarding disposal procedures.

#### 6. Warranty

- The warranty for the pipe N-P233 is valid for a period of one year from the date of purchase.
- Note that if this product is failed or damaged due to improper use the warranty may become void even if it happens during the warranty period.
- Note that the warranty is not valid for failures and damage unless this product is used together with the intelligent knee joint.

#### 7. Technical data

- Weight: 144 g
- Overall length: 290 mm
- Diameter of pipe: 34 mm, 30 mm

This device complies with the European Medical Device Regulation (EU) 2017/745.

#### Manufacturer

Nabtesco Corporation  
 Accessibility Innovations Company  
 Assistive Products Department  
 35, Uozakihama-machi, Higashinada-ku Kobe 658-0024, Japan  
 Phone: +81-78-413-2724 ; Fax: +81-78-413-2725  
 URL: <https://welfare.nabtesco.com>

#### Authorized representative for EU countries

PROTEOR SAS  
 6 rue de la Redoute  
 21850 Saint-Apollinaire, France  
 Phone: +33 3 80 78 42 42 Fax: +33 3 80 78 42 15  
[cs@proteor.com](mailto:cs@proteor.com)

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